

ANVERSO

ULKOX® OLE

PRODUCT DESCRIPTION

ULKOX® OLE is a hydrogel which contains Fucocert® (polysaccharide) and glycerine as moisturizers, as well as Olea europaea leaf extract, included because of its antioxidant properties. The presence of Carbopol 980® provides an isolating and protective barrier for the ulcer bed. The combination of these components, together with the capability to reduce the alkalinity of the ulcer bed through its slightly acidic pH, gives ULKOX® OLE the ability to modulate the environment of the cutaneous ulcers, thus promoting and speeding up the healing process.

COMPOSITION

Fucocert®, Glycerine, Carbopol.980®, Olea europaea leaf extract, Geogard Ultra®, disodium EDTA, Trietanolamine and purified water.

INDICATIONS

ULKOX® OLE is indicated for wound healing of full and partial thickness skin ulcers of different origin such as pressure ulcers, lower limb ulcers (venous ulcers), diabetic foot and anal fissures.

ULKOX® OLE can be used throughout the whole healing process. For skin ulcers It is recommended to be used at least three times per week coinciding with the regular cleaning of the ulcer and/or with the change of secondary dressing, if any, aiming to facilitate a favourable environment for the healing. For anal fissures It is recommended to make applications after bowel movements and personal grooming; as a rule, apply several times a day.

INSTRUCTIONS FOR USE

ULKOX® OLE hydrogel application:

For skin ulcers:

- Before the application, clean the ulcer and its edges with an appropriate cleansing solution and dry thoroughly.
- Unscrew the tube cap.
- Keeping the cannula away from the ulcer surface, gently press the tube and apply a 2-3 mm thin layer over the ulcer. In case of cavitated ulcers, apply the product up to 5 mm of thickness. Never exceed the surrounding area.
- Cover the area with a secondary dressing, if required.
- Once the secondary dressing is applied, remove any excess product from the surrounding area with a sterile gauze.

For Anal and/or rectal use:

- Before application of the product clean the affected area with warm water and mild soap and dry.
- Unscrew the tube cap.
- Apply the amount of gel to the anal region with one finger, using the tip of your finger to overcome the sphincter resistance. Wash your hands thoroughly before and after it.
- If the product should be applied inside the rectum, insert the canula of the tube carefully. Then apply a small amount of gel by putting a slight pressure on the tube. After use, carefully clean the cannula and re-cover it with the cap.

ULKOX® OLE is a multidose product: the content of the same tube can be used for several applications on the same ulcer until finished.

WARNINGS AND PRECAUTIONS

- Not for use in pregnant women, during breastfeeding period and in children.

REVERSO

- Can be used on lesions with clinical signs of infection, under a physician supervision.
- When used in skin ulcers, the mouth of the tube (cannula) must not come into contact with the lesion.
- Each tube must be used for a single patient to avoid cross-contaminations.
- Do not use the product after the expiry date printed on the package.
- Once opened, do not use after 6 months.
- Do not apply the gel (or remove it if it has been applied) in the event of a change in colour or bad odour.
- Do no use the product if there are any signs of tube damage and/or the cap is broken.
- For topical use only. Not for oral use. Product applications close to the eyes require extreme care.
- In case of any reaction in the application area, or any adverse event, stop usage and consult your physician.

CONTRAINDICATIONS

- It must not be used in patients with any known hypersensitivity to any gel ingredients.

STORAGE CONDITIONS

- Keep the package closed. Store at a temperature below 25°C. Do not freeze.

PRESENTATIONS

Presentations available:

- carton box containing 1 or 5 tubes of 15 g with integrated cannula for its application.
- carton box containing 1 tube of 30 g with integrated cannula for its application.

DATE OF TEXT REVISION FOR THE INSTRUCTIONS OF USE. Rev. 01: 11.02.2021

Noventure S.L.
Avenida Diagonal 549, 5ª planta
08029 Barcelona - Spain

STERILE R



Name: ULKOX OLE	DIMENSIONS (mm)	PRINTABLE COLOURS / PANTONES:	ELEMENT TO ADD TO THE PACKAGE:
PRESENTATION: Leaflet	Length: Width: 164 mm Height: 183 mm	Black	
MANUFACTURER:	Crease (mm) Width: 41 mm Height: 91,5 mm		
	NON PRINT: Die cut		
CREATED BY: Thinkinpress DATE: 11/02/2021 PROOF: 03			

Plegado: 2 veces mitad V + mitad H

ANVERSO

